

Circuit Alarm

Installation Manual on other brands



MODUFlex^{TM/MC}
BY DISPOMED

ASSEMBLY INSTRUCTIONS TO OTHER BRANDS

Adding an alarm to other brands

Parts required:

1 x [987-0010-000 Circuit alarm](#)


1 X 850-PLC22004 CPC male plastic
(Optional for adding open circuit)



VMS side view.




View of the alarm connection.

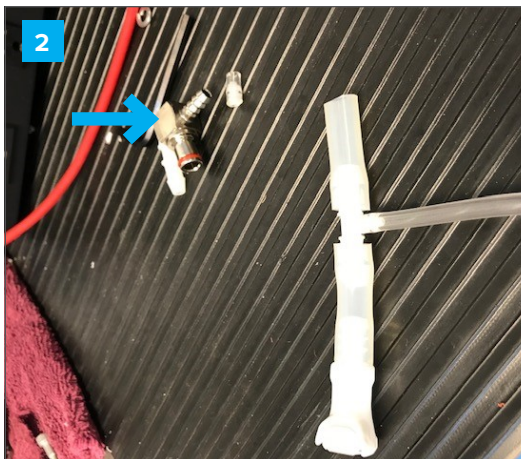

Remove metal connector from "T" on alarm.

ASSEMBLY INSTRUCTIONS TO OTHER BRANDS

MEDUFlex™

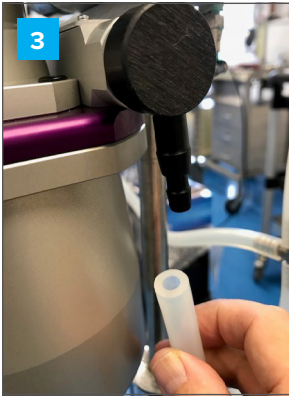


Connect the tube (the one we removed the connector from) to the outlet of the vaporizer.



Install metal connector on tube previously removed from the vaporizer and connect to alarm.



Remove one bolt from the vaporizer and use it to attach the smallest bracket. (A longer bolt may be necessary.)

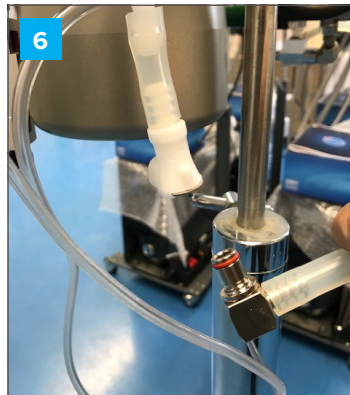

If you use another circuit (Bain, or other open circuits) you will need a second male connector. Attach the second male connector to your circuit and connect the circuit to be used in the female connector.



D I S P O M E D

MEDUFlexTM

Manufactured by Dispomed Ltd

Thanks for purchasing a Dispomed product.
For further assistance, contact customer support.

745 Nazaire-Laurin Joliette (QC) Canada J6E 0L6
T. 1-800-363-1746 (Canada and U.S.A only)
T. 1-450-759-9395
F. 1-450-759-8181

info@dispomed.com
www.dispomed.com

Printed in Canada

