

11. Reset the filter counters as follows:

a. Press the **On/Standby** key. The Replace Filter screen appears (*Figure 5-5*).

Figure 5-5. Replace Filter Screen



b. Press the **Replace Filter** key. The Filter Information screen appears (*Figure 5-6*).

Figure 5-6. Filter Information Screen - Filter Expired



c. Press the **Filter Reset** key.

d. Press the key code as described in **Service Screen Access*** on page 8-2. The filter counters are reset to 365 days and 2000 hours.

e. Press the **On/Standby** key to return to Standby mode. Press the **On/Standby** key again and verify that the Main screen appears (*Figure 5-7*) rather than the Replace Filter screen.

Service Screen Access* The warming unit provides screens for performing functions restricted to service personnel. These screens are protected by a key code. When you see the message “Enter Key Code” press the keys indicated in

Figure 8-1 simultaneously to access these screens.

Figure 8-1. Key Code Screen

